

◀ TEAR HERE

TEAR HERE ▶

CELOX™ -A

Applicator with Celox Hemostatic Granules

6g (0.2oz) Applicator

Temporary Traumatic Wound Treatment
To Control Moderate-To-Severe Bleeding
Promotes Rapid Coagulation

Works in Hypothermic Conditions*
Clots Heparinized Blood*

For Emergency Temporary External Use Only

FOR USE BY TRAINED EMERGENCY RESPONDERS

Instructions for Use on Back



Do not re-use



Do not re-sterilize

STERILE R

Sterilized using irradiation



Caution



Do not use if package is damaged

Rx Only

*CELOX Hemostatic Granules have been proven in controlled in-vivo trials to stop major arterial bleeding by coagulating the blood within 3 minutes. Laboratory testing has shown the product to be effective in blood temperatures down to 56°F (hypothermic). CELOX granules are also effective in blood containing Heparin. Data on file at Medtrade Products Ltd.

TEAR HERE

◀ TEAR HERE

TEAR HERE ▶


Instructions for Use:




1. Identify the point of bleeding and apply direct pressure. Swab excess blood where practical.



2. Tear open CELOX-A packet and select applicator.

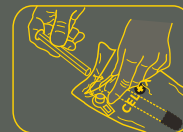

3. To prepare the applicator, remove clear plastic tab from the top of the applicator and insert plunger into applicator barrel.

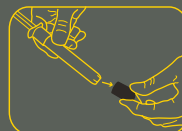

4. When ready to apply, remove blue end cap from the applicator.



5. Use the applicator to apply the granules to the bleeding site.



6. Slowly push plunger in while withdrawing the applicator and release granules to fill above skin level. Additional applicators or granule packs may be used if necessary.



7. Cover and apply FIRM pressure directly to the wound for 5 minutes. If bleeding persists, apply additional pressure until bleeding stops.



8. Secure with a dressing to maintain pressure on the wound. Discard any unused product.



9. Transfer patient to a medical facility as soon as possible.

10. Show empty pack to medical personnel.

ATTENTION MEDICAL FACILITY PERSONNEL

1. Physically remove CELOX Granules from the wound and any loose surface granules.
2. Fully flood entire wound area with sterile saline irrigation solution.
3. Proceed with normal irrigation and / or suction.
4. Ensure all product is removed from the wound prior to initiation of wound treatment.

Warnings: Not indicated for junctional wounds not amenable to pressure. The bleeding site must be visualized prior to application of Celox Granules; blind application to a bleeding site should be avoided. Device is not intended for surgical use. Do not use in abdominal wounds and wounds not amenable to pressure. Loss of sterility potentially poses a risk of infection. Do not apply over eyes. If eye irritation occurs flush with water for 5 minutes. Contains chitosan from shellfish. Allergy studies show no adverse reaction to date. Data on file at Medtrade Products Ltd.



Manufactured by:
Medtrade Products Ltd.
Electra House, Crewe Business Park,
Crewe, CW1 6GL, UK.
www.celoxmedical.com



FG08832071
Artwork Ref: MTP-22-2493
Issue Date: 31/Oct/2022

CELOX™ is a trademark of Medtrade Products Ltd. All rights reserved.