

◀ TEAR HERE

TEAR HERE ▶

# CELOX™

## Hemostatic Granules

Temporary Traumatic Wound Treatment  
To Control Moderate-To-Severe Bleeding  
Promotes Rapid Coagulation

Works in Hypothermic Conditions\*  
Clots Heparinized Blood\*

For Emergency Temporary External Use Only

### 15g (0.5oz)

FOR USE BY TRAINED EMERGENCY RESPONDERS

Instructions for Use on Back



Do not  
re-use



Do not  
sterilize

STERILE R

Sterilized using  
irradiation



Caution



Do not use if  
package is  
damaged

Rx Only

\*CELOX Hemostatic Granules have been proven in controlled in-vivo trials to stop major arterial bleeding by coagulating the blood within 3 minutes. Laboratory testing has shown the product to be effective in blood temperatures down to 56°F (hypothermic). CELOX granules are also effective in blood containing Heparin. Data on file at Medtrade Products Ltd.

◀ TEAR HERE

TEAR HERE ▶

Instructions for Use:



**1.** Identify the point of bleeding and apply direct pressure. Swab excess blood where practical.



**2.** Tear open CELOX packet.



**3.** Immediately pour entire contents of pouch to fill the wound above skin level. Additional packs may be used if necessary.



**4.** Cover and apply FIRM pressure directly to wound for 5 minutes. If bleeding persists, apply additional pressure until bleeding stops.



**5.** Secure with a dressing to maintain pressure on the wound. Discard any remaining CELOX granules.

**6.** Transfer patient to Medical Facility as soon as possible.

**7.** Show empty pack to medical personnel.

ATTENTION MEDICAL FACILITY PERSONNEL

1. Physically remove CELOX Granules from the wound and any loose surface granules.
2. Fully flood entire wound area with sterile saline irrigation solution.
3. Proceed with normal irrigation and/or suction.
4. Ensure all product is removed from the wound prior to initiation of wound treatment.

**Warnings:** Device is not intended for surgical use. Do not use in abdominal wounds and wounds not amenable to pressure. Loss of sterility potentially poses a risk of infection. Do not apply over eyes. If eye irritation occurs flush with water for 5 minutes. Contains chitosan from shellfish. Allergy studies show no adverse reaction to date. Data on file at Medtrade Products Ltd.



Manufactured by:  
Medtrade Products Ltd,  
Electra House,  
Crewe Business Park,  
Crewe, CW1 6GL, UK.

Issue Date: 23 Jan 2020  
Artwork Ref: MTP-19-1420

**REF** FG08830551